Management
System
ISO 13485:2016
ISO 9001:2015
www.ferrox.com
tel. 800200544

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SINGLE WALL BAR IN BEECH WOOD 112262

DOUBLE WALL BAR IN BEECH WOOD 112264

INSTRUCTIONS FOR THE ASSEMBLY. CAUTION

The wall bar should be fixed to a brick or cement wall.

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 **ASSEMBLY BY PROFESSIONAL INSTALLER ONLY.**

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 **Read the instructions and use the product as directed according to its function.**

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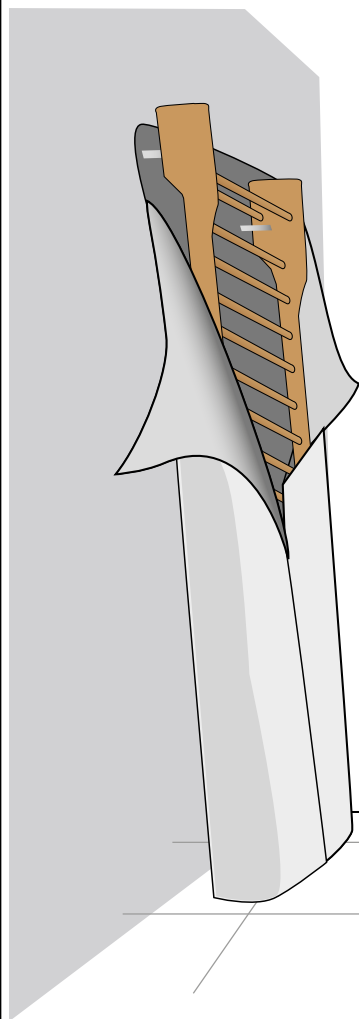
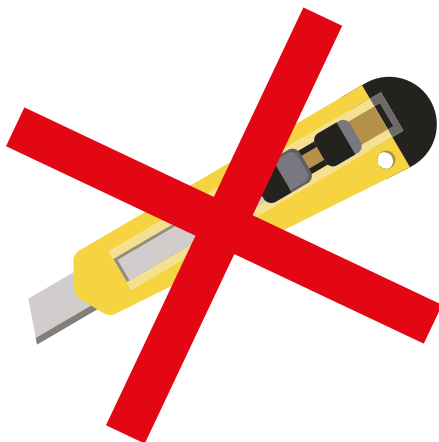
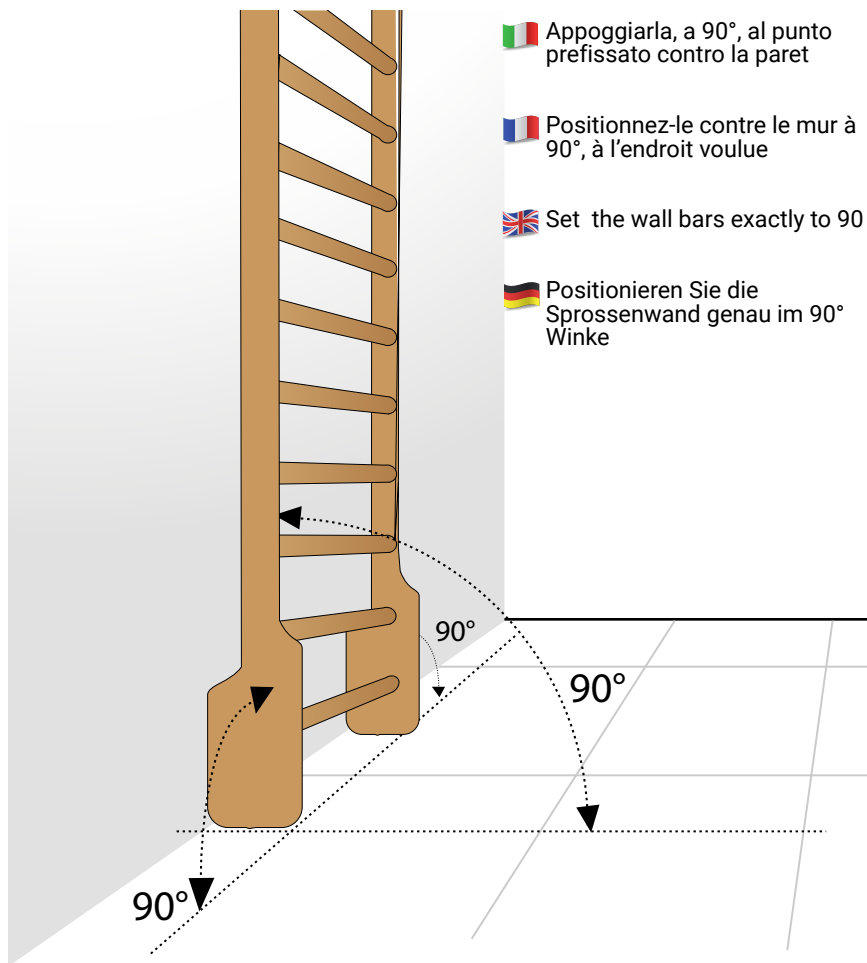
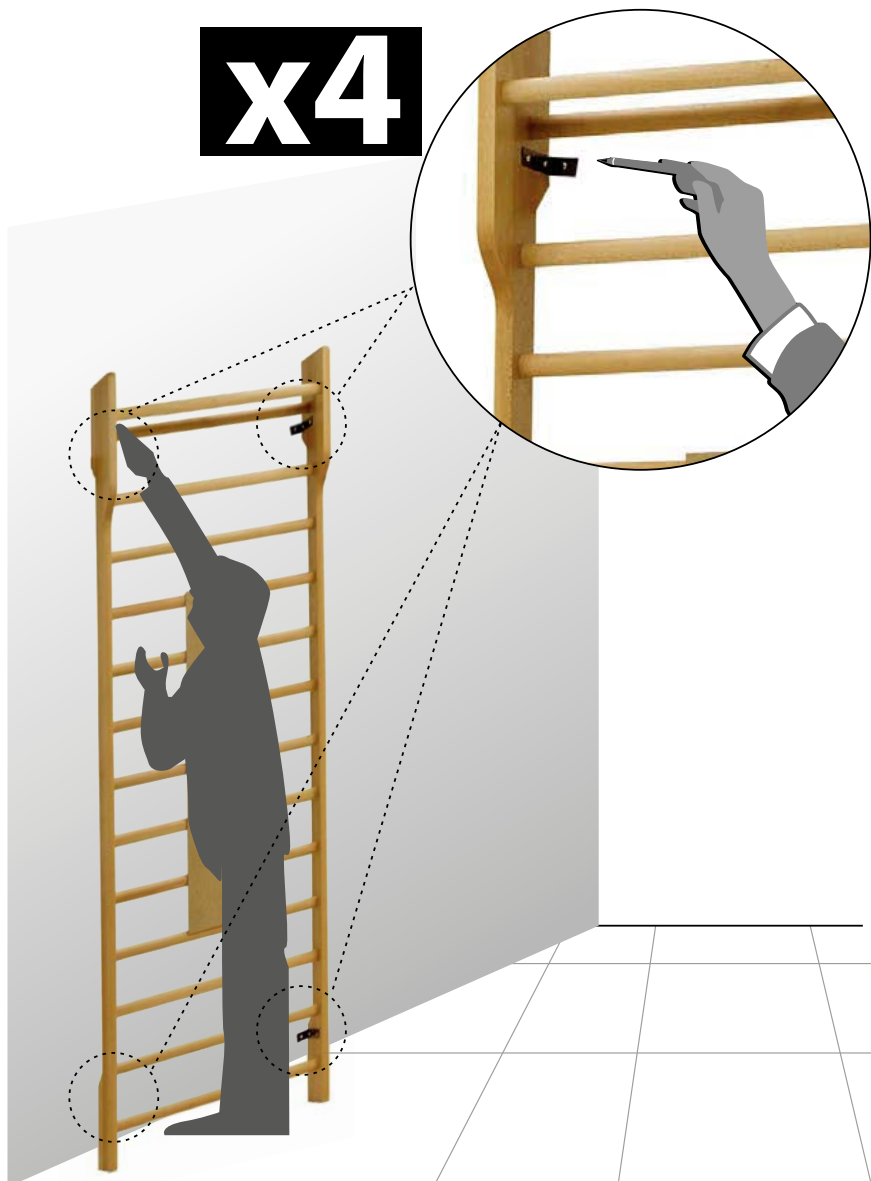
 Make sure that:
- The mounting brackets lie flat on the wall
- The bottom ends of the rungs touch the floor

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

 Mark the drillholes.

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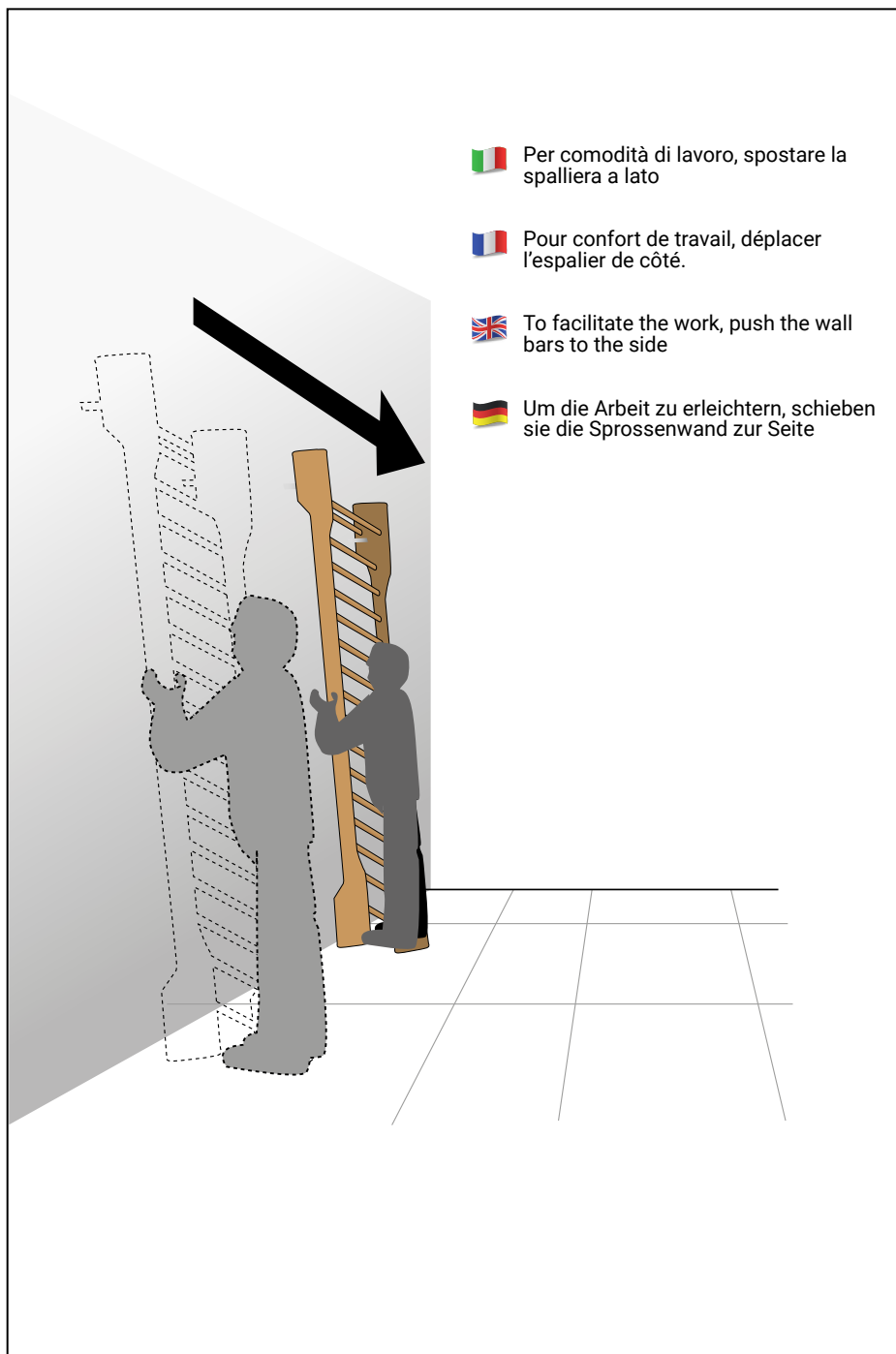
Technical drawing of a wall corner showing dimensions for a power tool. The wall is 2270 units high and 800 units wide. A power tool is shown plugged into a wall outlet. The distance from the bottom of the wall to the outlet is 225 units.

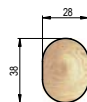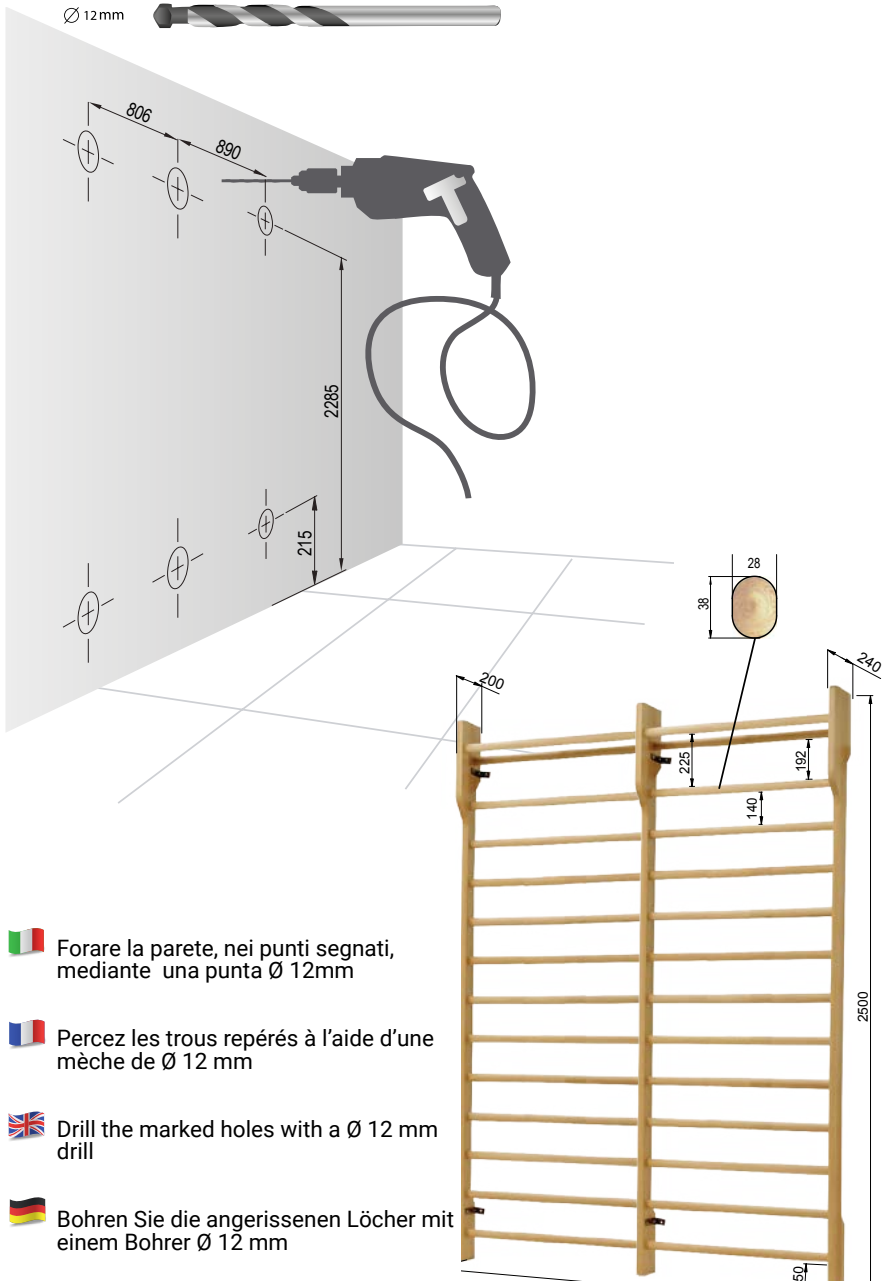-  Forare la parete, nei punti segnati, mediante una punta Ø 12mm
-  Percez les trous repérés à l'aide d'une mèche de Ø 12 mm
-  Drill the marked holes with a Ø 12 mm drill
-  Bohren Sie die angerissenen Löcher mit einem Bohrer Ø 12 mm

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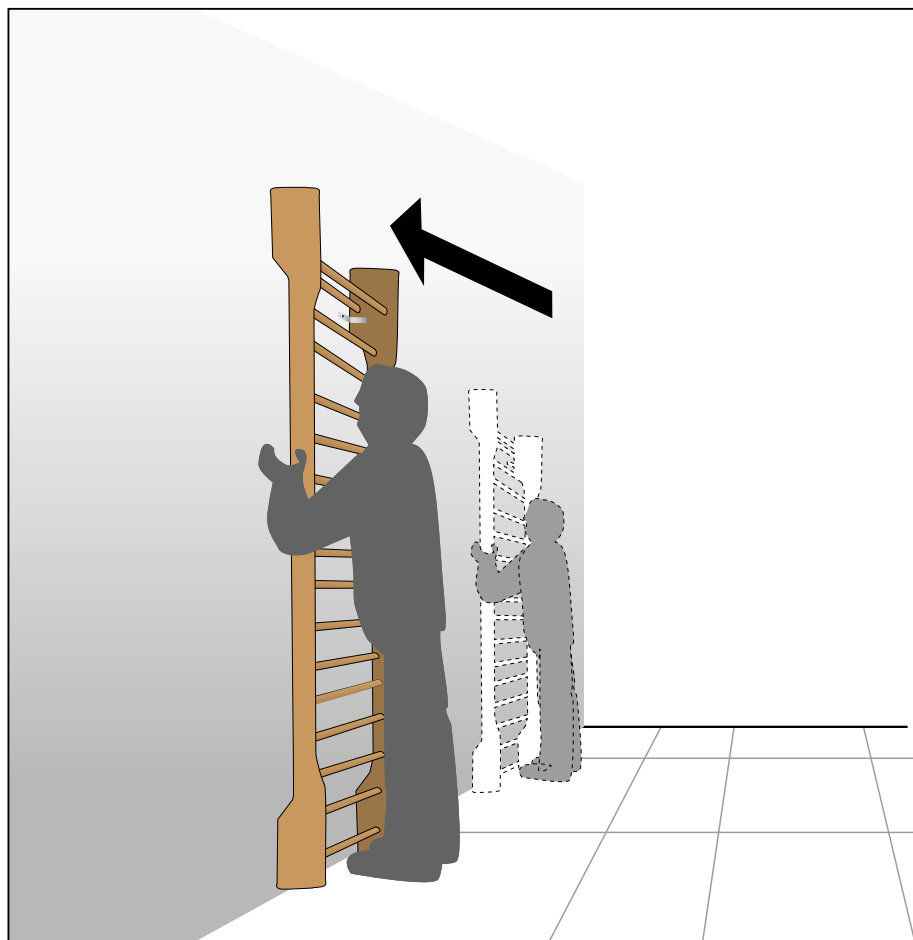
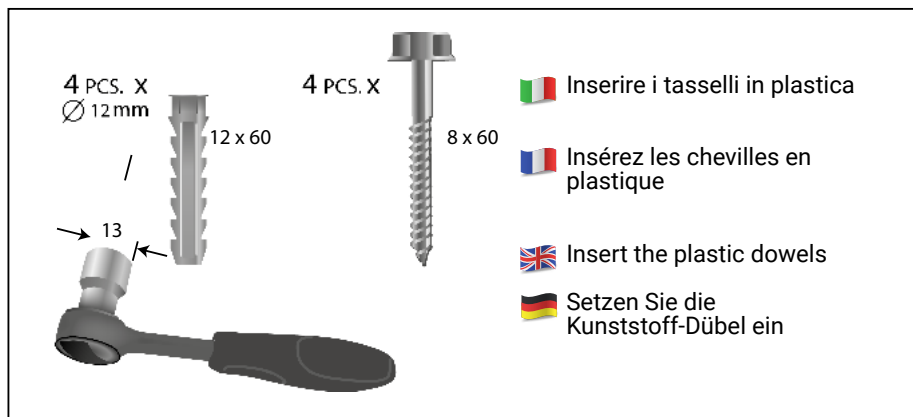
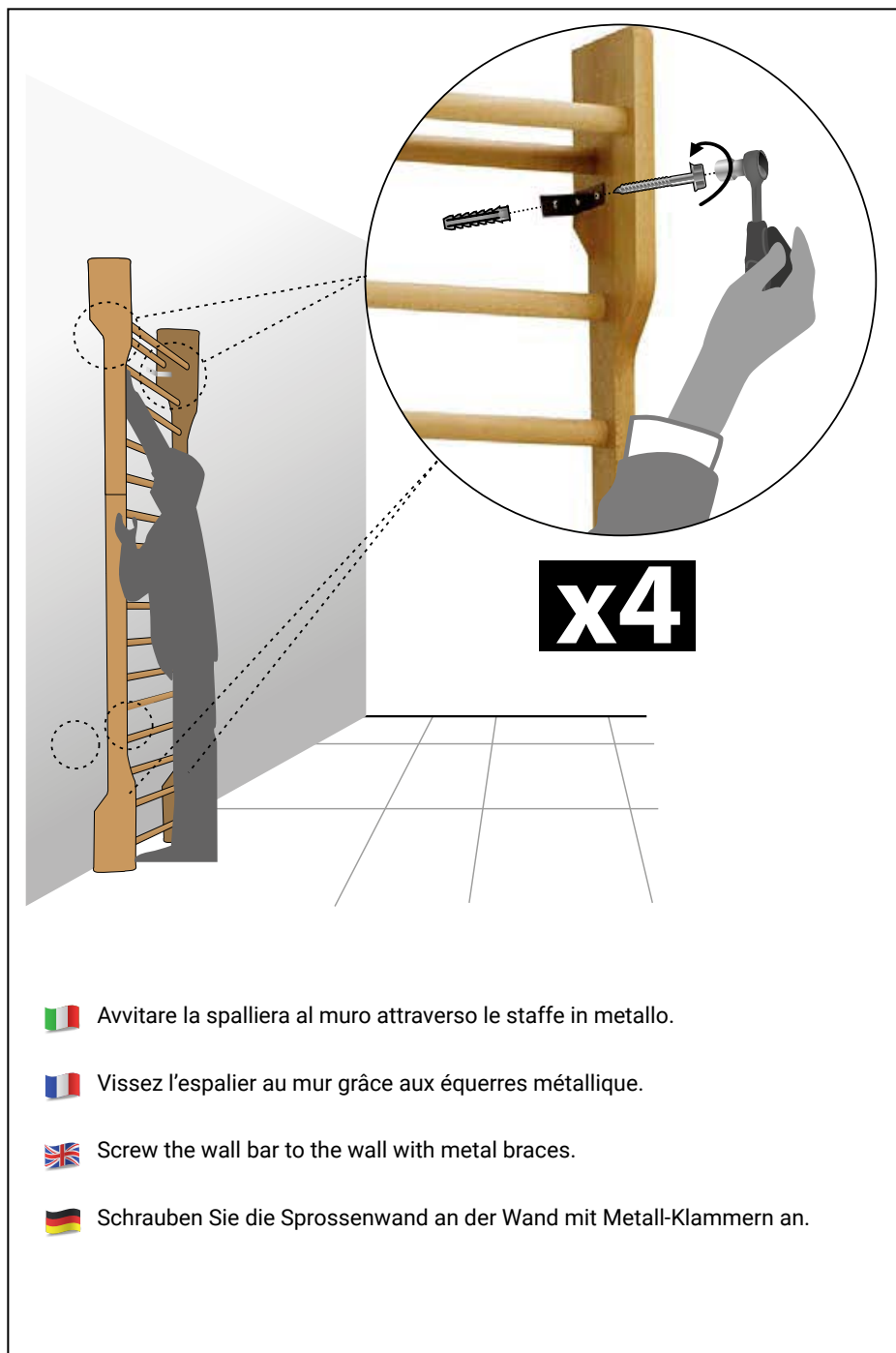
Drill the marked holes with a Ø 12 mm drill

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Screw the wall bar to the wall with metal braces.

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CLASSIFICATION AND USE

SINGLE WALL BAR IN BEECH WOOD DOUBLE WALL BAR IN BEECH WOOD 112262 - 112264

Read the instructions and use the product as directed according to its function.

Do not use the product in a way other than its intended purpose.

INTENDED USE

Multi-purpose wall bars for wallbar where it is possible to apply small accessories suitable for the realization of simple traction systems
Help the injured limb to recover normal joint movement.

TYPE OF PATIENT

Patients with non-significant rheumatic or traumatic pathologies.

PLACE OF USE

Doctors' surgeries and physiotherapy clinics.
Sports use.

OPERATORS QUALIFIED TO USE THE PRODUCT

All healthcare professionals, physiotherapists and similar.
All domestic or professional users.

The instructions given to the patient by the reference healthcare professional suffice.

CONTRAINDICATIONS

Not suitable for use in a way other than the intended purpose.

OTHER INDICATIONS

- It must be fastened to the wall by a professional.
- Do not exceed the workload indicated
- Check accessories (ropes and braces) for wear

**Report all serious incidents that occur due to the device.
Report the incident to the manufacturer and the competent authority**

At the end of its life, the product must be disposed of in accordance with local regulations and taken to specific waste collection facilities.

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SINGLE WALL BAR IN BEECH WOOD 112262 - DOUBLE WALL BAR IN BEECH WOOD 112264

Dimensions 112262: 90 X 250 cm **Dimensions 112264:** 180 X 250 cm

Weight 112262: 23 kg **Weight 112264:** 26 kg

Storage: dry, dark place, -20°+60°

Cleaning: it is generally recommended to clean the device after each use and to carry out the disinfection cycle periodically. To clean the device, wipe its parts with a cloth moistened with warm water. Do not use any agents that could damage the surface of the device through abrasion or overly aggressive treatment.

Operating temperature: 0° / +40°

Expected lifetime: it is estimated that the device can be used for 10 years.

It is mandatory for the user and the patient to use disposable gloves and shoes with clean rubber soles with disposable overshoes

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FERROX S.r.l.
Verantwortlicher **FERROX S.r.l.**